



Product information

Innodes IMP Impression Disinfection

2500 ml REF 5013K



Characteristics

- Effective in just **1 minute!** (Additional TB effectiveness after 15 min. also effective against noroviruses after 30 min.)
- Contains no aldehyde, alcohol or toxic ingredients – safe to use!
- Pleasant scent
- Licensed as a medical product

Product type

Ammonium based ready-for-use solution.

Areas of application

For the disinfection of all dental impression materials.

Application

Immerse the impression in question into the ready-for-use disinfection solution and remove it again after the recommended application time. The impression must be fully covered by the solution. Immerse alginates and hydrocolloids for a short time only and let them dry. Then rinse the impression with water. Do not mix with cleaning agents or other disinfectants. Depending on the degree of contamination, the working solution may need to be replaced (RKI recommendation: at least once per working day). The working solution can also be disposed of in the sewage water system.

Application times

Bacteria (incl. MRSA) as per VAH, EN 13727 and EN 14561 (dirty conditions):

Conc. 1 min.

Candida albicans as per VAH, EN 13624 and EN 14562 (dirty conditions):

Conc. 1 min.

Enveloped viruses (e.g. HBV, HCV and HIV) as per RKI/DVV (with conditions):

Conc. 1 min.

TbB (M. terrae in a suspension test, clean conditions):

Composition

100g contains:

0.24g Benzyl-C12-16-alkyl
dimethylammonium chloride

0.24g N,N-didecyl-N,N-dimethyl
ammonium chloride

0.24g C12-14-
alkyl[(ethylphenyl)methyl]dimethyl
ammonium chloride

0.35g chlorhexidine gluconate

Chemical-physical data

Density: 1g/cm³

pH value: 7.2

Conc. 15 min.

Noroviruses as per EN 14476, (clean conditions):

Conc. 30 min.

Rotaviruses as per BGA/DVV, (clean conditions):


Conc. 1 min.

Safety guidelines

H 412 Harmful to aquatic life with long lasting effects.

Avoid release to

the environment. Dispose of the contents/container in accordance with local/regional/national/international regulations.

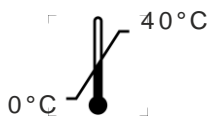

Certificates

Our products have been scientifically examined and tested in accordance with the latest methods. We will be happy to send you copies of our assessments on request.

Contact

PRISMAN Pharma International AG
Am Stalden 16
4622 Egerkingen
Switzerland

Tel.: +41 (0)62 206 20 60

Fax: +41 (0)62 206 20 69

www.innocid.com

info@prismanpharma.com

Manufacturer

PRISMAN GmbH
Otto-Hahn-Ring 6-18
64653 Lorsch
Germany